Made in Germany

 **tickSAFE**®
www.ticksafe.com

MED

ZECKENGREIFER
TICKGRIPPER
PRENEUSE DE TIQUE
AFFERRA-ZECICHE
QUITA-GARRAPATAS

SOFT - GRIP - TECHNOLOGY

Zecken sicher entfernen
Remove ticks safely
Enlever les tiques en sécurité
Rimuovere le zecche in modo sicuro
Retirar garrapatas de forma segura

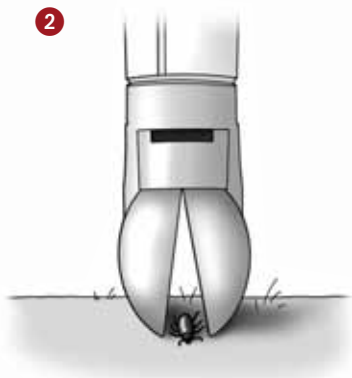
1 a


1 b

2

3

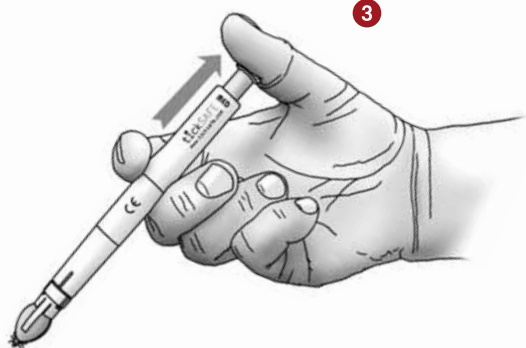4

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Operating Instructions

- 1 Grasp the tickSAFE® Tickgripper in the palm of the hand (1a, 1b) with the thumb on the push button and press the push button completely down to fully open the gripper head.
- 2 Carefully position the opened gripper head vertically to the skin surface so that the tick is positioned in the center of the two gripper tips.
- 3 Slowly release the push button. The gripper head should grasp the tick and begin to rotate.
- 4 The tick is removed and captured in the gripper head. Should the removal procedure be unsuccessful, repeat steps 1 to 4. Repeated attempts may be required for very small ticks!

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Squeezing a tick may promote infections!

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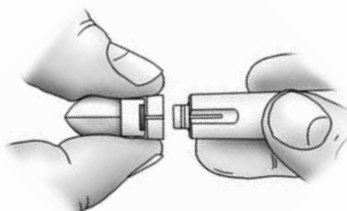
Congratulations

on the purchase of your new tickSAFE® Tickgripper! Conventional methods often lead to squeezing the tick and can therefore promote infections. The soft-grip technology minimizes this risk. Please observe the following operating instructions:

- ▶ Only use the Tickgripper to remove ticks! Do not misuse for other purposes!
- ▶ As the Tickgripper contains small parts which a child may swallow, keep such safely stored away from small children.
- ▶ Handle the Tickgripper carefully. Avoid flicking or playing with the push button. Such can damage the internal mechanical parts. The Tickgripper is not a toy and thus should not be handled by small children.
- ▶ Store the Tickgripper in such fashion that the gripper head is not squeezed or compressed.

Deformation can impair the precise closure of the gripper head, which is essential for the removal of small ticks. Replace the protective cap on the Tickgripper after use.

▶ Large ticks can rupture during removal and contaminate the Tickgripper with blood. In this event a change of the gripper head is recommended for hygienic purposes.

If you adhere to this advice, the Tickgripper will become an indispensable aid for tick removal.

Our General Business Terms and Conditions can be found at www.ticksafe.com.

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Useful accessories are available at www.ticksafe.com

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Exchange of the gripper head

To replace the gripper head slowly pull it off the clasp which is located between the shaft of the Tickgripper and the gripper head.

There will be slight resistance which is normal. To attach the new gripper head hold the Tickgripper shaft and gripper head aligned to each at the given positioning marks. Click the gripper head onto the clasp located on the Tickgripper shaft.

Important: Do not tilt, twist or jam! Remove or affix the gripper head only in the longitudinal axis of the Tickgripper shaft.

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tickSAFE GmbH